Installation manual

MANUEL D'INSTALLATION
INSTALLATIEHANDLEIDING
INSTALLATIONSANLEITUNG
MANUALE DI INSTALLAZIONE
MANUAL DE INSTALACIÓN
INSTRUKCJA INSTALACJI
PŘÍRUČKA PRO INSTALACI

Club

CE

Preface	4
1. This product	5
2. Scope of delivery	6
3. Transport	6
3.1. Loading the transport trolley	6
4. Assembling the bed	11
4.1. Tools	11
4.2. Assembly	12
4.3. Emergency lowering of the headrest	20

Preface

To support you on the installation and repairs of this nursing bed, we provide you with this installation manual. Please read it carefully. If you still have questions after reading this manual, do not hesitate to contact Vermeiren.

The information in this manual applies to the following bed(s):

- Club

Important note

Pictures of the product are used to clarify the instructions. Details of the depicted product may deviate from your product.

Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual For user and specialist dealer
	Installation instructions For specialist dealer
	Service manual for beds For specialist dealer
	Drawings of (spare) parts For specialist dealer
	EC declaration of conformity

1. This product

EN

1. Head panel
2. Lifting pole / Trapeze handle (optional)
3. Mattress retainer
4. Foot panel
5. Side rail (optional)
6. Castor wheel with brake
7. Foot section
8. Head section
9. Handset
10. Identification plate

2. Scope of delivery

The following components are part of the delivery:

- Head and feet end incl. motors
- Slatted frame, foot-end incl. motor
- Slatted frame, head-end incl. transformer and motor
- Handset
- User manual
- Options

Before use, check if everything is included and that no products are damaged.

3. Transport

CAUTION

Risk of injuries or damage

- Attach all bed parts and transport trolley firmly to the vehicle, to prevent them from moving during transport.

3.1. Loading the transport trolley


CAUTION

Risk of injuries or damage


- Check if all parts are well secured.

3.1.1. Vertical transport set (Club ; Club Vario)


3.1.1.1. Club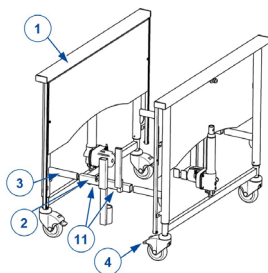

1. Grasp the panels (1).
2. Mount the attachment of the transport set (2) in the attachment of the panels (3).
3. Lock the brakes of the panels (4).
4. Fasten the panels in place with the star knobs (11). (If required, use and tighten the nut on the star knobs to counteract any potential unscrewing during transport.)

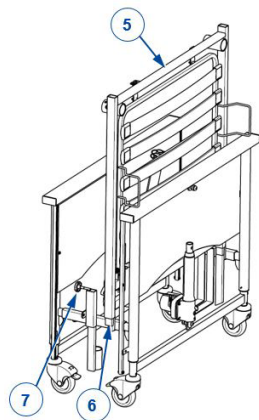
5. Grasp the head end (5) and hold this with the motor located to the bottom side.
6. [Label Nr. 1] Slide the tube of the head end (5) in the tube of the transport set (6). Secure the head end (5) with the star knobs (7).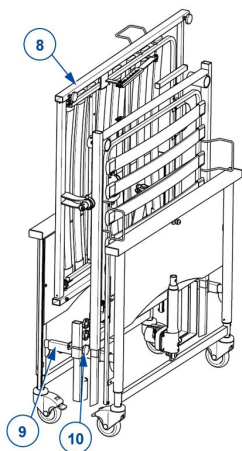


7. Grasp the foot end (8) and hold it sideways with the motor located on the inside.
8. [Label Nr. 2] Place the foot end on the transversal tubes (9) of the transport set and tighten with the star knob (10).
9. Secure the slatted frames with the hook-and-loop strap. Attach the strap to the top tube of one frame part by looping it through the buckle, and then wrap it around the top tubes of both frame parts.


3.1.1.2. Club Vario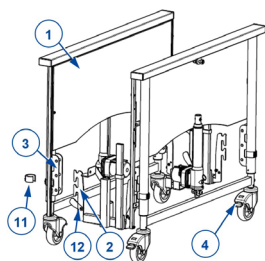

1. Grasp the panels (1).
2. Make sure that the red locking levers (12) on the transport kit are in the unlocked position (leaning towards the inside).
3. Mount the attachment of the transport set (2) in the attachment of the panels (3) and lock with the locking clamps (11). Mount the transport set in the lowest position of the panels.
4. Lock the panels in place by turning the red locking levers (12) toward the panels.
5. Lock the brakes of the panels (4).

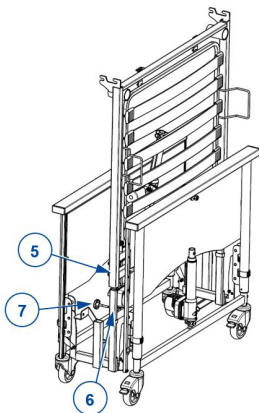
6. Grasp the head end (5) and hold this with the motor located to the bottom side.
7. [Label Nr. 1] Slide the tube of the head end (5) in the tube of the transport set (6). Secure the head end (5) with the star knobs (7).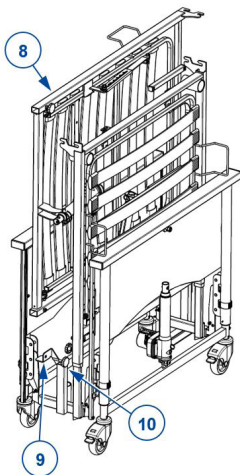



8. Grasp the foot end (8) and hold it sideways with the motor located on the inside.
9. [Label Nr. 2] Place the foot end on the supports (9) of the transport set and tighten with the star knob (10).
10. Secure the slatted frames with the hook-and-loop strap. Attach the strap to the top tube of one frame part by looping it through the buckle, and then wrap it around the top tubes of both frame parts.

3.1.2. Horizontal transport set (universal)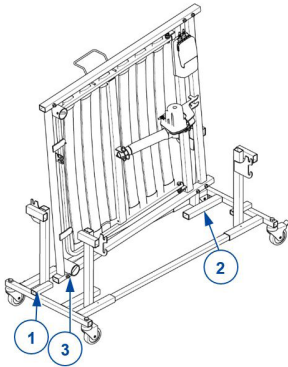

1. Grasp the head end and hold it with the motor to the inside of the transport set.
2. Slide the bed frame tube over the short tube (1) on the transport set. Let the other end rest on the long transport set tube (2).
3. Secure the head end with the star knob (3).
4. Repeat the steps above for the foot end.

5. Grasp the head and foot panel.
6. For Club Vario, Club UL: Mount the panel attachments (4) to the hooks (5) of the transport set (for Club Vario: use the lowest attachment). Lock with the locking clamps.
7. For Club: Mount the panel attachments (6) to the short tube (7) of the transport set. Tighten with the screw.
8. Repeat for the other panel.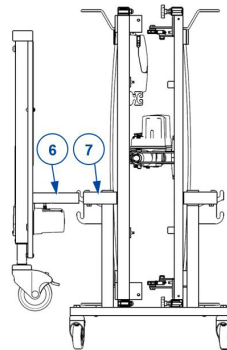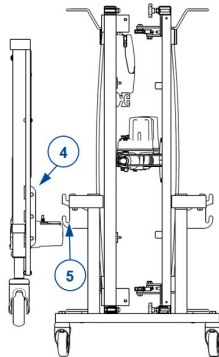

4. Assembling the bed

CAUTION

Risk of injuries or damage

- The bed needs to be (dis)assembled by a specialist dealer according to the instructions in this chapter.
 - Only use the settings described in this manual.
 - Only use parts described in this manual.
 - Only use genuine spare or replacement parts approved by Vermeiren.
 - Prevent bystanders from accessing the working area.
 - Keep the (dis)assembling area clean and tidy, to prevent falling.
 - Collect all small parts in a box or bag.
 - While (dis)assembling and operating the bed, make sure no objects or body parts get caught between parts. In particular, pay attention to the position of the cables.
-  Read the user manual of this bed. Be aware of the technical details and limits of intended use. For more information, contact Vermeiren.

4.1. Tools

To set up the bed, the following tools are needed:


- Wrench no. 6
- Allen key no. 4, 5

4.2. Assembly

4.2.1. Head panel / foot panel

4.2.1.1. Club


1. Slide the attachment tube of the head panel (4) into the tube of the slatted frame (3).
2. Secure the slatted frame (1) with the star knobs (2).
3. Repeat for the foot panel. Make sure that the slatted frames are mounted securely.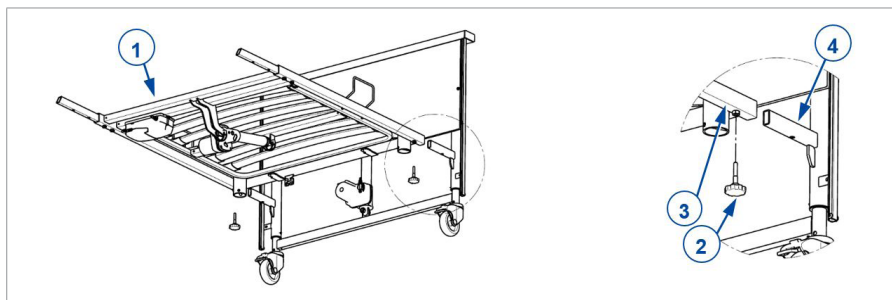

4.2.1.2. Club Vario

1. Mount the attachment of the slatted frame (3) on the attachment of the head panel (4).
2. Secure the slatted frame (1) with the locking clamps (5).
3. Repeat for the foot panel. Make sure that the slatted frames are mounted securely.


The slatted frame can be mounted in 2 different positions (B, C): step 64 mm.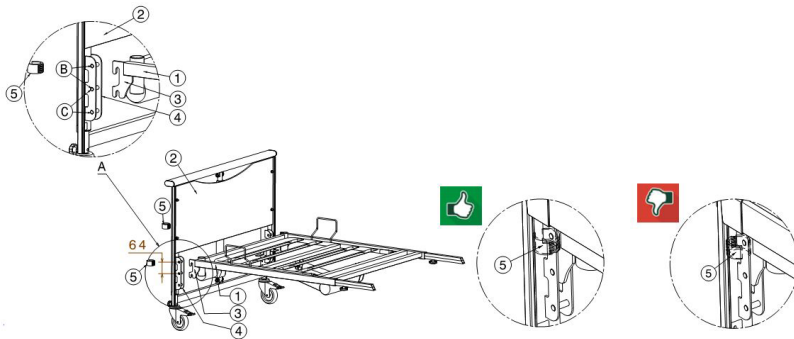

1. Slatted frame
2. Head and foot panels
3. Attachment slatted frame
4. Attachment panels
5. Locking clamp


4.2.1.3. Club *UL*

1. Mount the attachment of the slatted frame (3) on the attachment of the head panel (4).
2. Secure the slatted frame (1) with the locking clamps (5).
3. Repeat for the foot panel. Make sure that the slatted frames are mounted securely.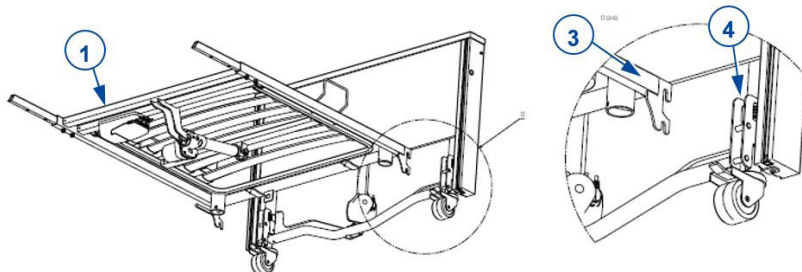

4.2.2. Slatted frame


1. Remove the star knobs (2).
2. Insert the connector (3) into the frame tubes of the foot and head slatted frame. Make sure it is oriented correctly: the holes in the inner and outer tubes have to align to insert the star knobs, see image.
3. Insert and retighten the star knobs (2) of the slatted frame connection.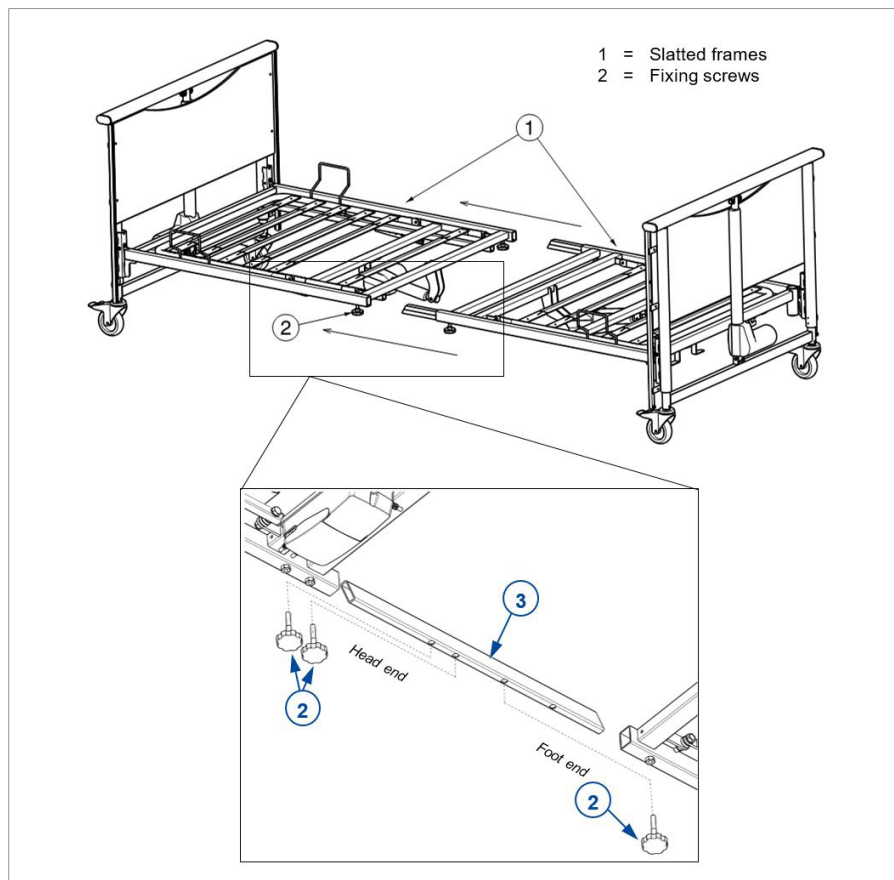

4.2.3. Side rails

4.2.3.1. Wooden side rails


Adjust the bed to medium height. If the bed is adjusted too low, it is not possible to mount the side rails; if it is adjusted too high, you have to pull the side rails too far upwards.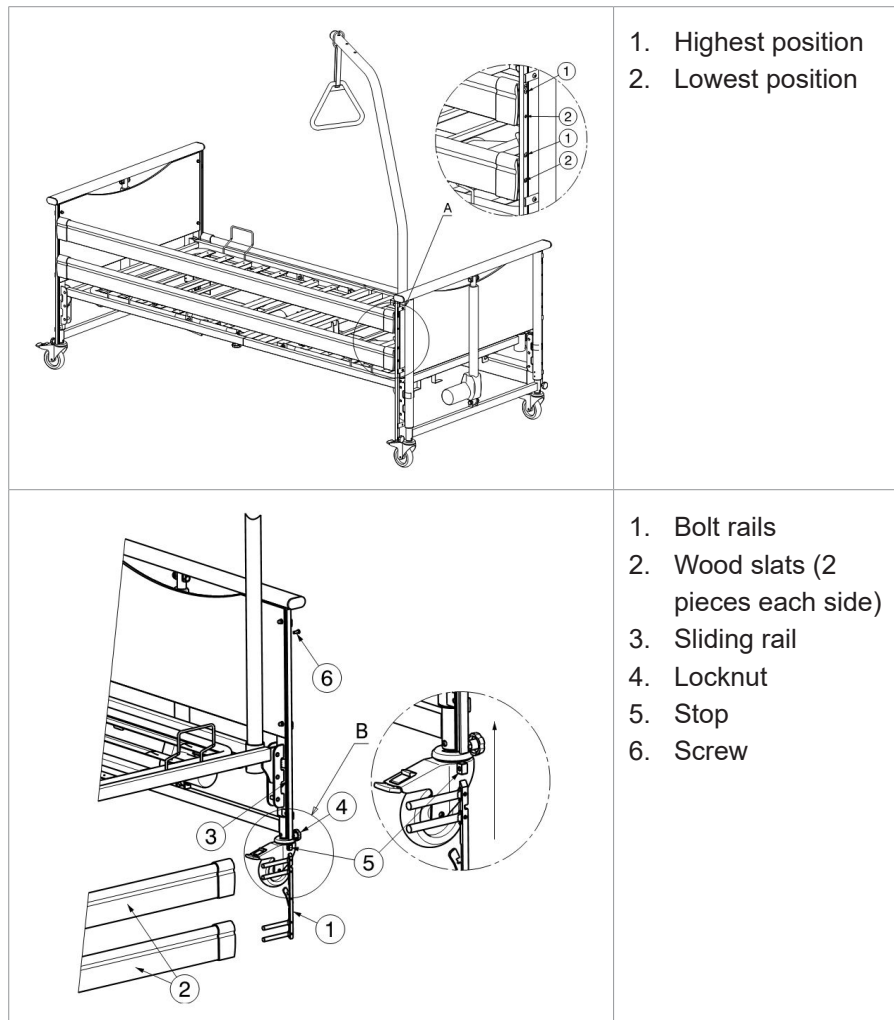

1. Insert the bolt rails (1) on the right and left in the upper wood slat (2).
Insert the bolt rails (1) right and left in the lower wood slat (2).
2. Slide the entire bolt rails (1) + wooden slats (2) in the relevant sliding rail (3) from the head and foot panel. Grasp the upper wooden slat and pull upwards until the wooden side rails lock in position.
3. Insert the bolts (4) and tighten them.
4. Repeat these steps for the other side rail.


4.2.3.2. Metal side rails (not for Club UL)

1. Place the hooks (1) over the frame tube (2).
2. Locate the arrow sticker on the inside of the head-side slatted frame.
Position the side of the hook (1) against the arrow head.
3. Tighten the star knob (3). Make sure that both sides have been tightened securely.
4. Repeat these steps for the other side rail.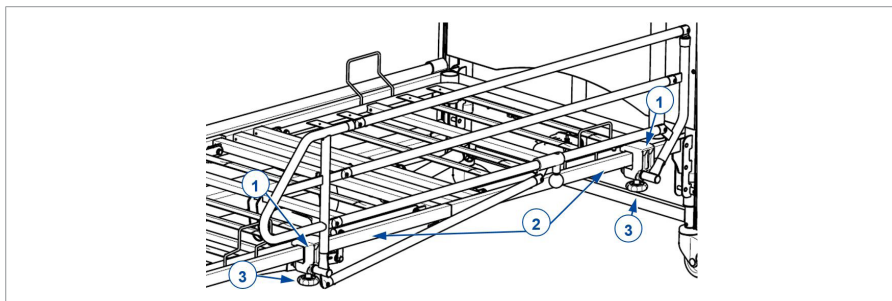

4.2.3.3. Split side rails (not for Club UL)

- i** Mind the difference between head-end and foot-end rails (head-end rails are shorter than foot-end rails).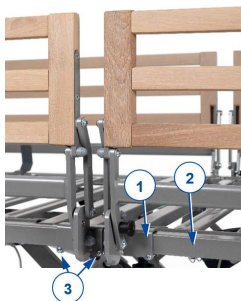


1. Take one of the split side rails.
2. Place the split side rail attachments (1) over the mattress frame support (2).
3. Firmly tighten the two screws (3) on the bottom of each attachment.
4. Repeat for all other split side rails.
5. Make sure that all side rails are firmly attached into place.

4.2.4. Transformer connections

CAUTION

Risk of injuries

- Do not place any objects on the connection and the cable, and protect them against mechanical loads.
1. Connection requires a 220V/230V, 50Hz power socket conform to VDE specifications.
 2. The diagram for the transformer connections is given in the figure below. To insert the cables, open the top lid of the transformer. Otherwise the cables will not be secured into place.
 3. When all cables have been secured, close the top lid.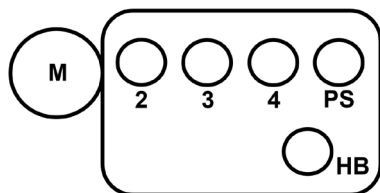

1. M = Head section motor
PS = Power supply
HB = Handset
2. Connection of head panel motor
3. Connection of knee-bending angle motor
4. Connection of foot panel motor

- i** The motor connection plugs are labelled with the same number as the connections on the transformer.

Take hold of the plug only to remove it from the power socket.

The power connection must not have any loops or kinks and should not pass around sharp edges. Protect the power connection and the cable leading to the handset from being crushed.

4.2.5. Strain relief and power cable


⚠ CAUTION

Risk of damage


- The power cable must be carefully positioned without kinks, without pulling, and without touching the floor.

Below the cross bar of the head slatted frame, there is a strain relief hook for the power cable.

1. Pull the desired length of the power cable in such a way that the power cable can be laid under the slatted frame without sagging. Be careful to avoid that the cable comes under traction in any adjustment position.
2. Fasten the power cable to the strain relief: make a loop around the top of the strain relief (1), then loop each side of the cable behind the small hooks.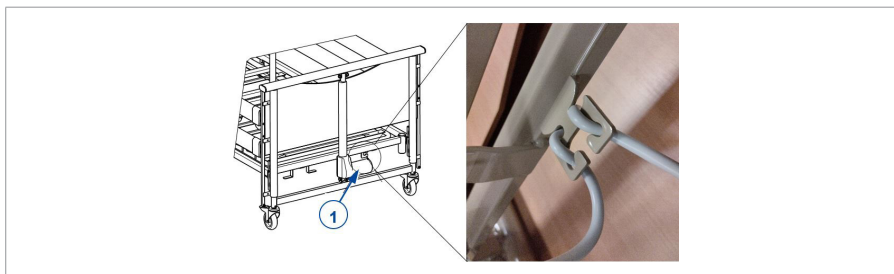

4.2.6. Cable placement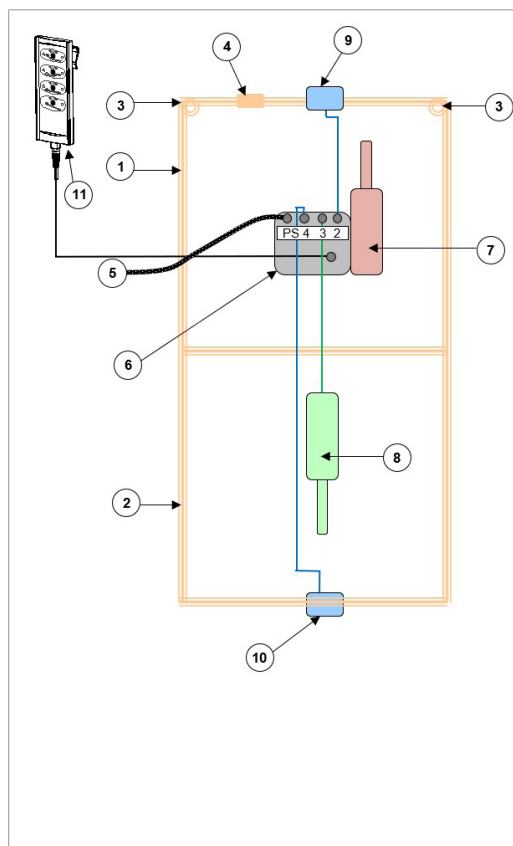


1. Mattress slatted frame head end
2. Mattress slatted frame feet end
3. Sockets lifting pole
4. Strain relief
5. Power supply
6. Control box
7. Head end motor
8. Knee-bend motor
9. Head panel motor
10. Feet panel motor
11. Handset

All wires within the bed frame should be placed in such a way that contact with the floor, deformity and crushing are avoided. Insert the power cables for the motors in the opening on the motor mounts.

4.2.7. Lifting pole (optional)

The lifting pole can be mounted on either side of the head end.

1. Insert the pole (1) as far as it will go into the relevant socket (2).
2. Let the groove on the lifting pole (3) slide over the tube in the socket.
3. Pull the strap loop over the end of the upper frame (4) until the desired position.
4. Pull the strap (5) through the triangle (6). Fix its length and secure the strap.

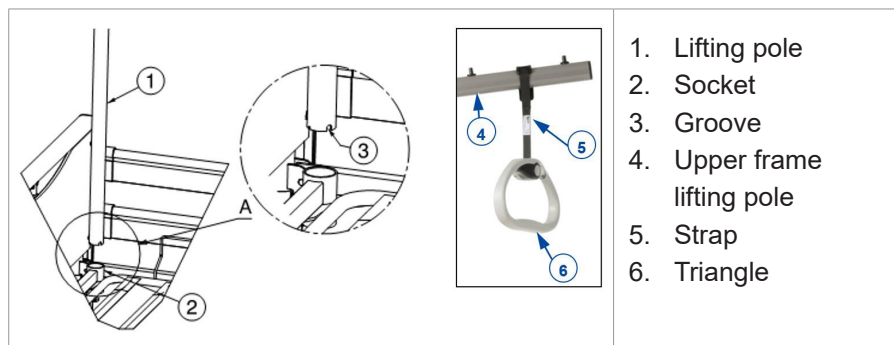4.3. Emergency lowering of the headrest

⚠ CAUTION

Risk of injuries

- Make sure that there are no objects or body parts between the headrest and the frame.
- Emergency lowering is only permitted if at least two persons are available.

The headrest can be lowered manually (emergency lowering in the case of motor failure):

1. Firmly grasp the headrest to prevent it from falling.
2. A second person can release the suspension pin by folding the securing batch and pulling the pin from the motor suspension.
3. Lower the headrest gently.

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All rights reserved, including translation.