



Use glue H900009 /
Uzywac kleju H900009

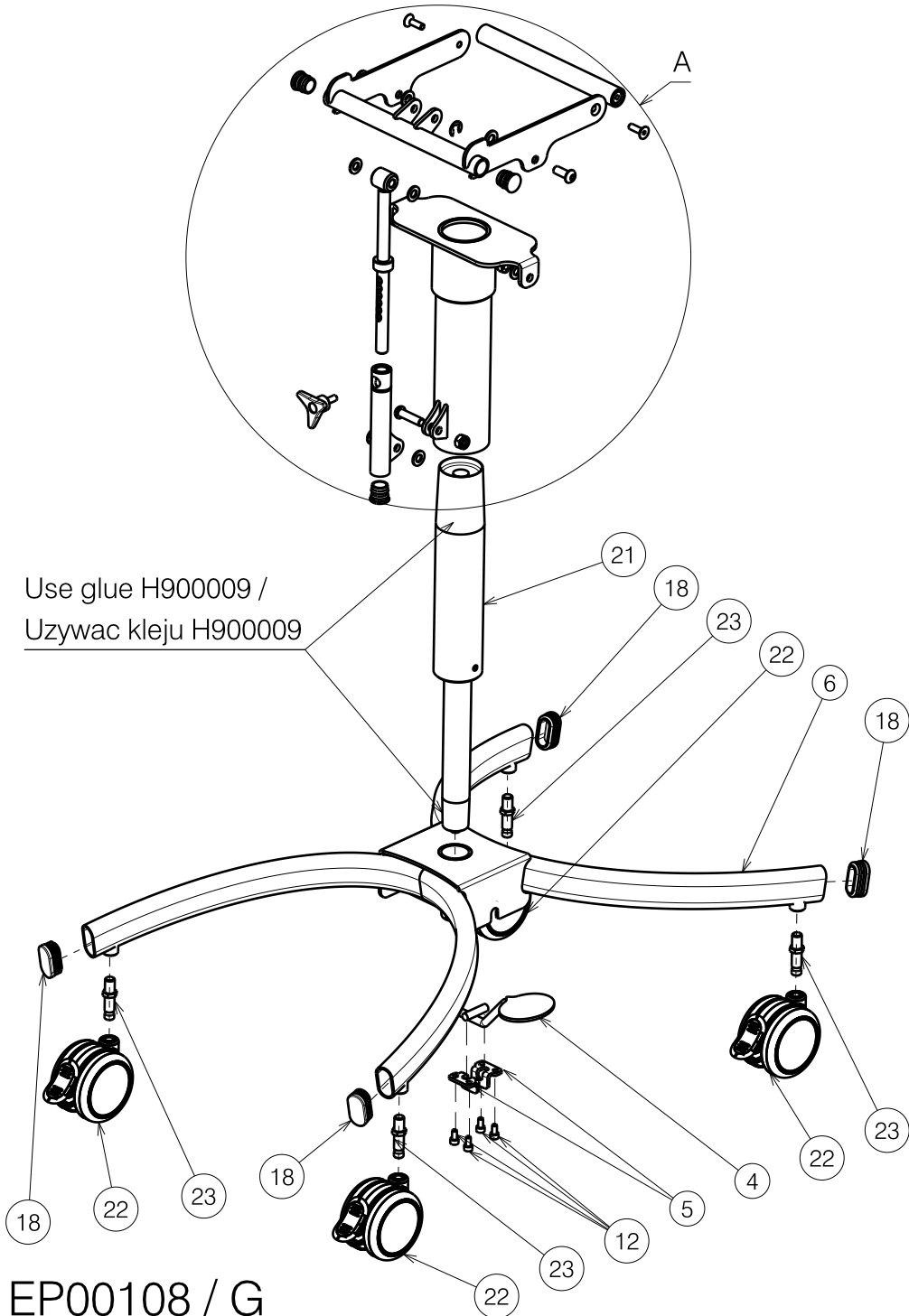

EP00108 / G

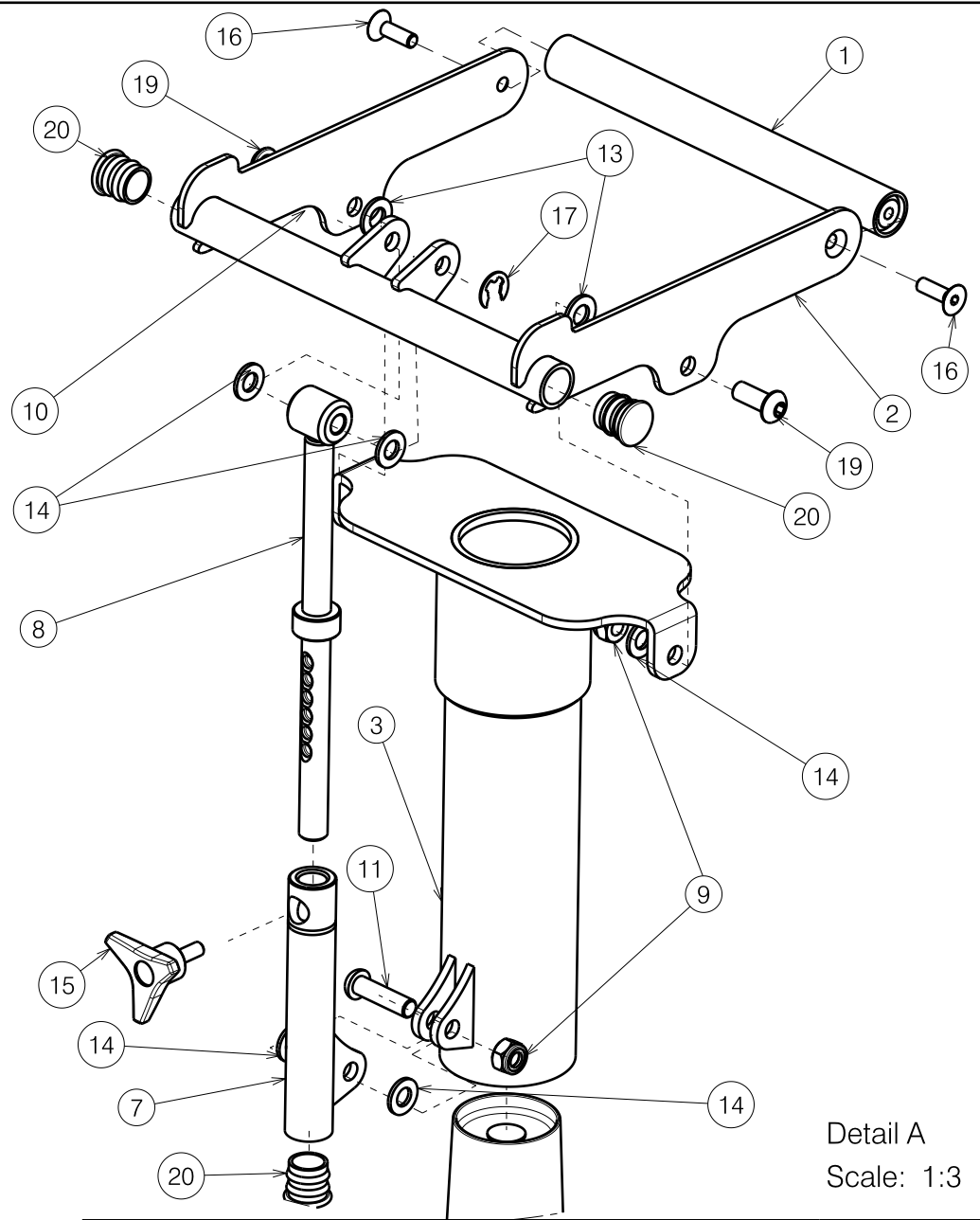

Detail A
Scale: 1:3

LOWER FRAME U82 SIZE 32				Plan No.
				EP00108/ G
Sheet 1/2				
Scale: 1:7	Drafted	Verified	Confirmed	
Date	04/02/2013	23/11/2018	23/11/2018	
Initials	WM	PM	TK	



Material			
No.	Plan	Article	Quantity
1	YP00521	1816000	1
2	YP04116	2846500	1
3	YP04121	2846106	1
4	YP04126	2846254	1
5	YP04131	2851500	2
6	YP04135	2846306	1
7	YP04140	2846406	1
8	YP04141	2846600	1
9	X001016	1001682	3
10	XP01112	2871800	1
11	M8x25	1990501	1
12	M5x10 DIN912	1001488	4
13	D9xD17 DIN126	1001554	2
14	D8.4xD17	1003199	6
15	M6x15	1009932	1
16	M6x20 DIN7991	1903081	2
17	---	1904152	1
18	---	1906679	4
19	M8x20	1992349	2
20	---	2990095	3
21	---	2992512	1
22	---	2992513	4
23	---	2992514	4

EP00108 / G

LOWER FRAME U82 SIZE 32				Plan No.
Sheet 2/2				EP00108/ G
Scale: 1:7	Drafted	Verified	Confirmed	
Date	04/02/2013	23/11/2018	23/11/2018	
Initials	WM	PM	TK	