



1. Way of assembly with side support

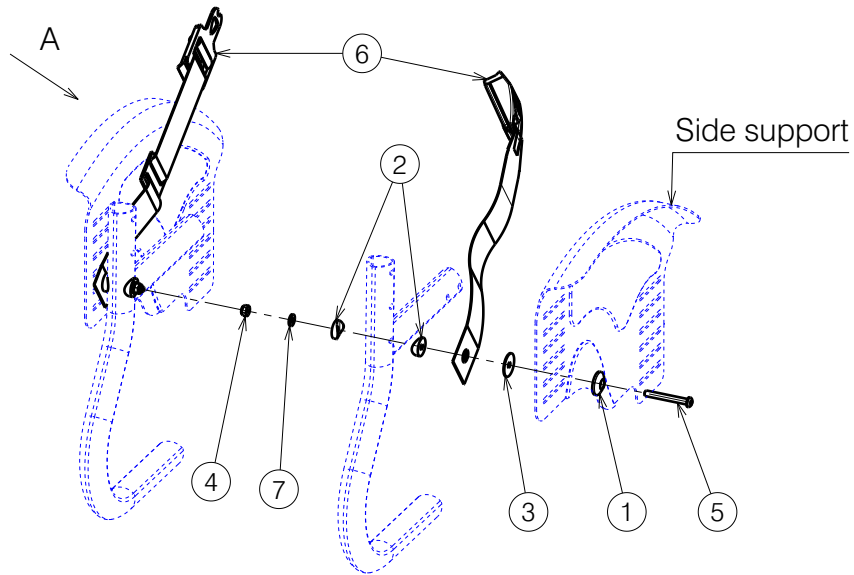


View A

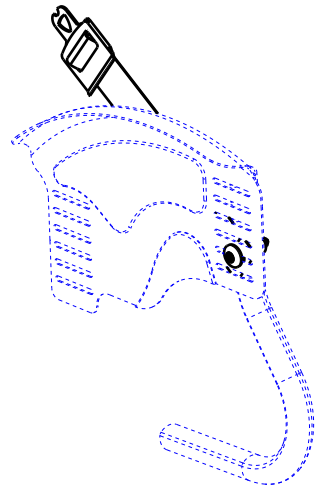


Bill of Material 1			
No.	Plan	Article	Quantity
1	XP00366	2990331	2
2	XP01560	2993561	4
3	X003036	1997486	2
4	M6 DIN985	2994063	2
5	M6x50	2994965	2
6	B58	1992267	1
7	D6,4 DIN125	2994414	2


2. Way of assembly without side support

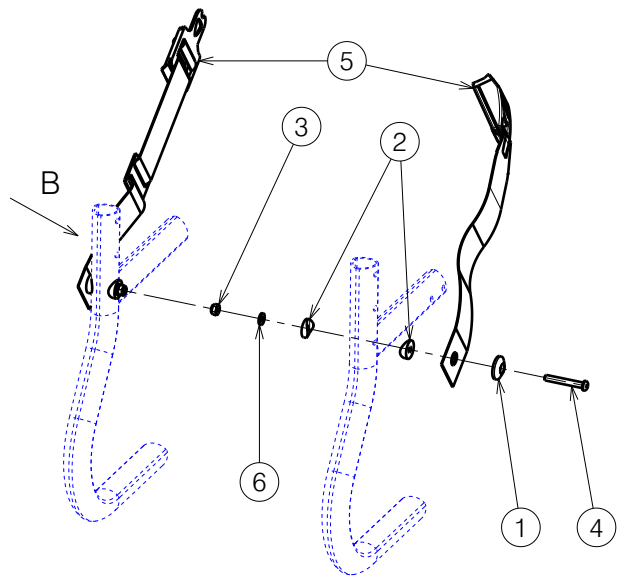


View B

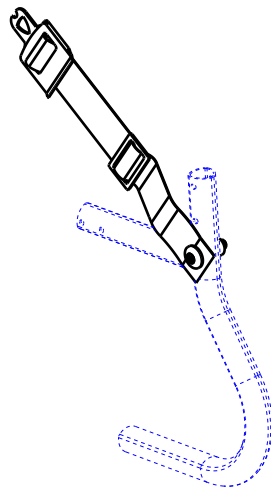


Bill of Material 2			
No.	Plan	Article	Quantity
1	XP00366	2990331	2
2	XP01560	2993561	4
3	M6 DIN985	2994063	2
4	M6x45	2994086	2
5	B58	1992267	1
6	D6,4 DIN125	2994414	2

<h2>B58 FOR TRIGO</h2>				---
				Plan No. E00035434/A.1
Scale: 1:8	Drafted	Verified	Confirmed	
Date	2022-08-10	10/08/2022	10/08/2022	
Initials	AZ	AS	AZ	

3. Way of assembly with side support

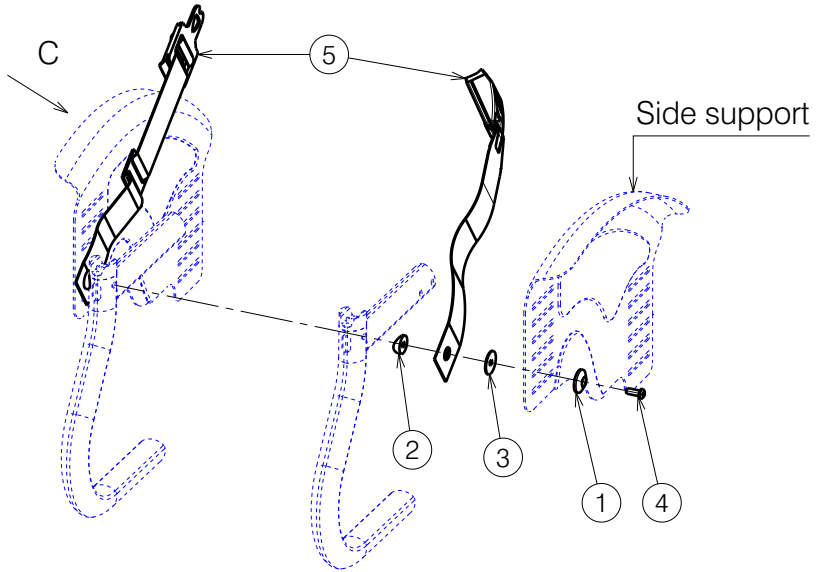


View C

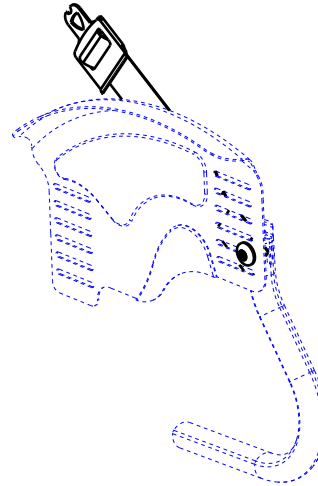


Bill of Material 3			
No.	Plan	Article	Quantity
1	XP00336	2990331	2
2	XP01560	2993561	2
3	X003036	1997486	2
4	M6x16	2994067	2
5	B58	1992267	1


4. Way of assembly without side support

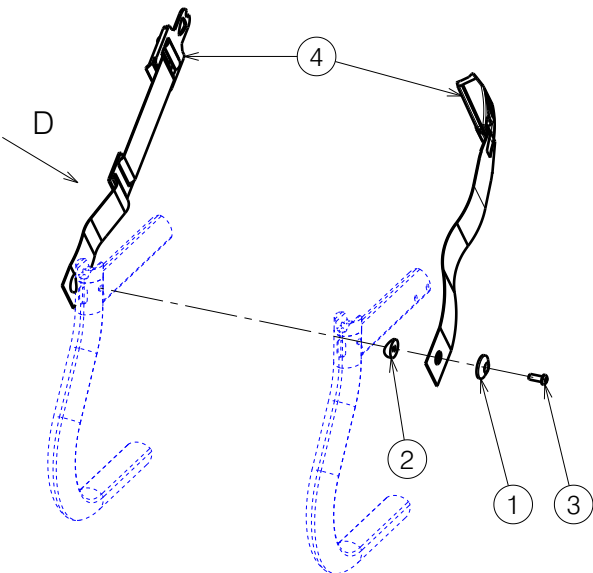

View D

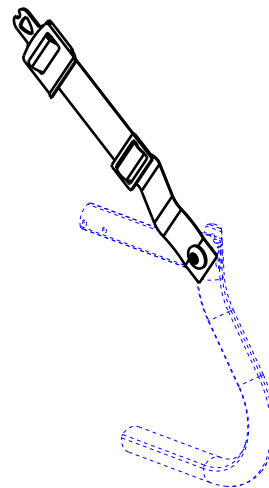

Bill of Material 4			
No.	Plan	Article	Quantity
1	XP00366	2990331	2
2	XP01560	2993561	2
3	M6x16	2994067	2
4	B58	1992267	1

B58 FOR TRIGO				---
2/2				Plan No.
				E00035434/A.1
Scale: 1:8	Drafted	Verified	Confirmed	
Date	2022-08-10	10/08/2022	10/08/2022	
Initials	AZ	AS	AZ	