



L70 always with headrest L55 /
L70 zawsze z zagłówkiem L55

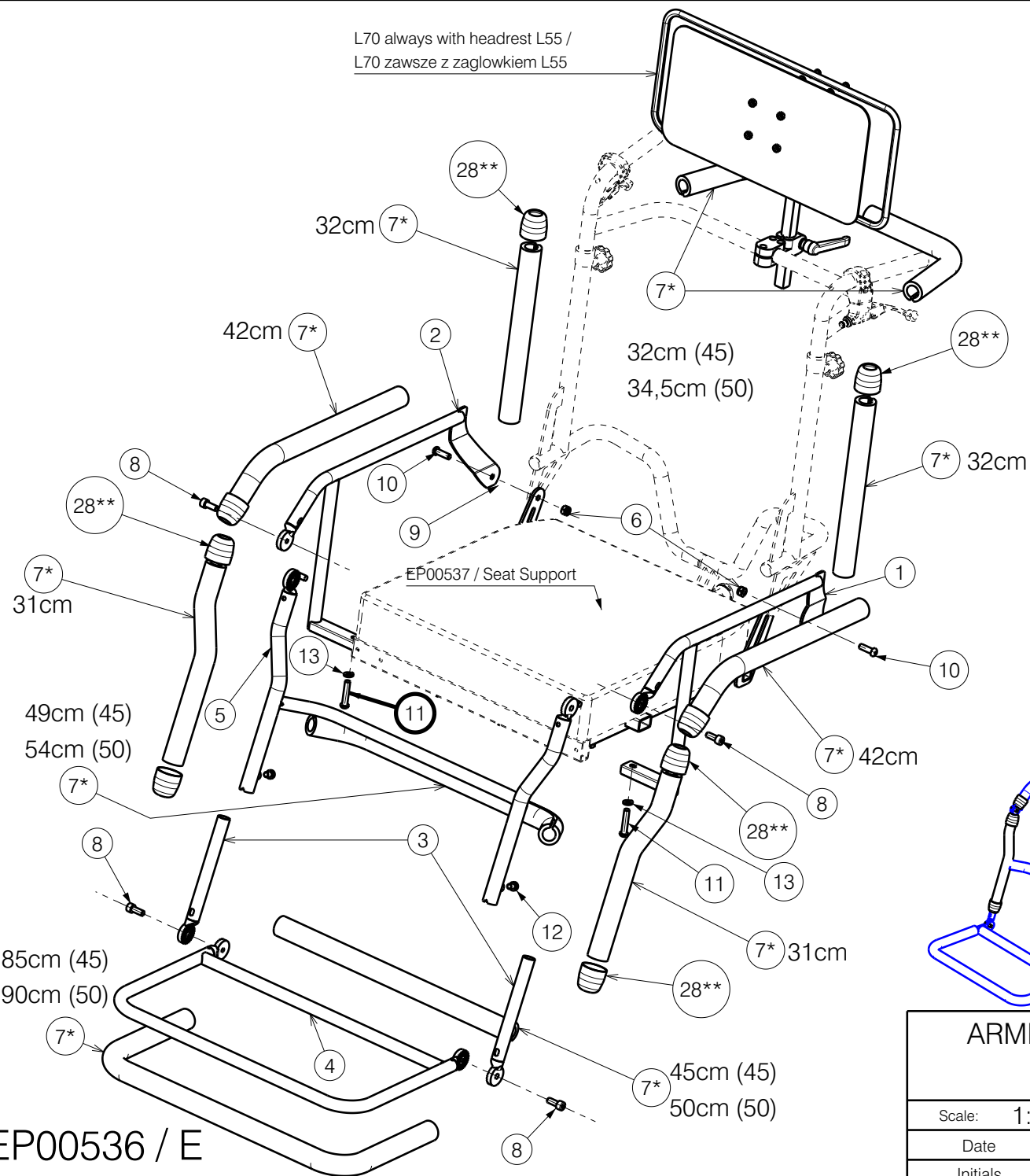

| Material | | | |
|----------|-----------------|---------|----------|
| No. | Plan | Article | Quantity |
| 1 | YP08412 | 2615725 | 1 |
| 2 | YP08413 | 2615825 | 1 |
| 3 | YP08423 | 2616100 | 2 |
| 4 | YP08427 (45) | 2616325 | 1 |
| | YP08431 (50) | 2616425 | 1 |
| 5 | YP08666 (45) | 2624625 | 1 |
| | YP08667 (50) | 2624725 | 1 |
| 6 | X001016 | 1001682 | 2 |
| 7* | --- | 1992923 | 12 |
| 8 | M8x20 DIN912 | 1906425 | 4 |
| 9 | D8,4 DIN 125 | 1003199 | 2 |
| 10 | M8x25 ISO 7380 | 1990501 | 2 |
| 11 | M8x40 ISO 7380 | 1008739 | 2 |
| 12 | M8x10 DIN912 | 1903926 | 2 |
| 13 | M8 DIN 127B | 2991295 | 2 |
| 14 | M6x25 DIN 7984 | 2994897 | 2 |
| 28** | SHRINKDOWN HOSE | T001045 | 0,64m |

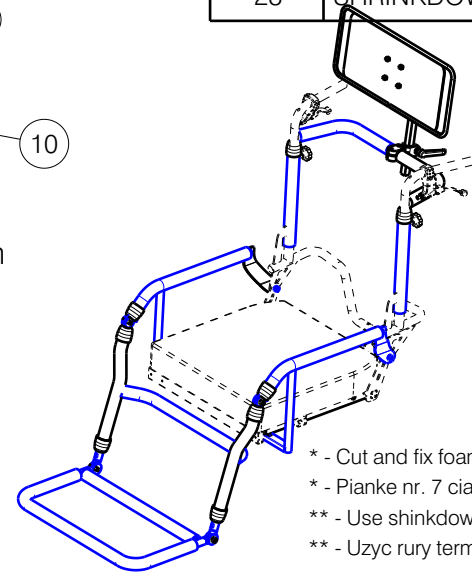

- * - Cut and fix foam no. 7 by black electrical tape /
- * - Pianke nr. 7 ciac i mocowac przy pomocy czarnej tasmey izolacyjnej
- ** - Use shrinkdown hose at the ends. /
- ** - Uzyc rury termokurczliwej na zakonczeniach.


EP00536 / E

ARMREST AND FOOTREST SUPPORTS /
GRANULATING SYSTEM L70 Sheet 1/3

Plan No.
EP00536/ E

| | | | |
|------------|------------|------------|------------|
| Scale: 1:9 | Drafted | Verified | Confirmed |
| Date | 03/08/2020 | 03/09/2021 | 03/09/2021 |
| Initials | AZ | PM | TK |



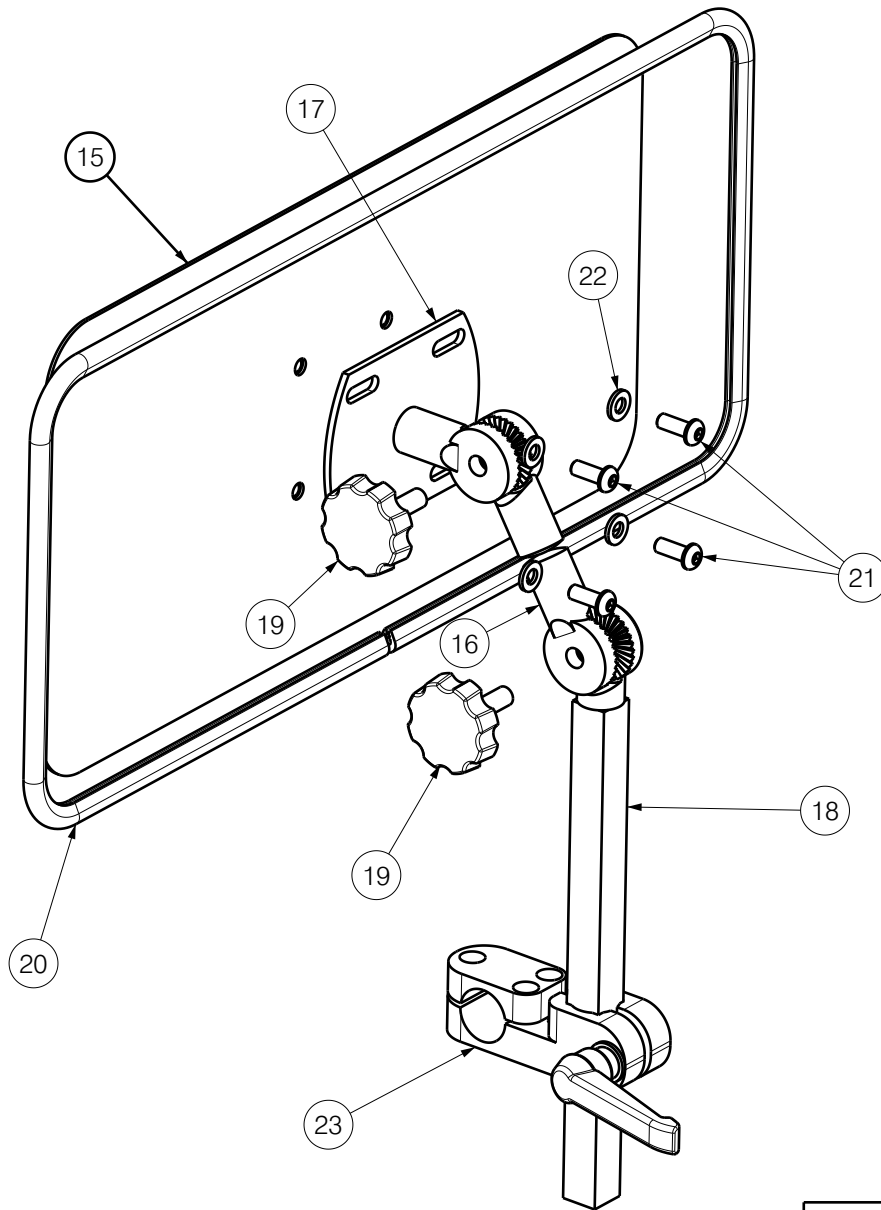

Isometric view
Scale: 1:3

| Material | | | |
|----------|-----------------|---------|----------|
| No. | Plan | Article | Quantity |
| 15 | YP08996 | 2634354 | 1 |
| 16 | Y003413 | 1463254 | 1 |
| 17 | Y003414 | 1463354 | 1 |
| 18 | Y007779 | 1607954 | 1 |
| 19 | 1001580StarKnob | 1001580 | 2 |
| 20 | 1003299 | 1003299 | 1,3m |
| 21 | M6x16 ISO 7380 | 1008459 | 4 |
| 22 | M6 DIN 125 | 1001564 | 4 |
| 23 | EP00216 | CLAMP | 1 |

ARMREST AND FOOTREST SUPPORTS /
GRANULATING SYSTEM L70 Sheet 2/3


Plan No.
EP00536/ E

| | | | |
|------------|------------|------------|------------|
| Scale: 1:9 | Drafted | Verified | Confirmed |
| Date | 03/08/2020 | 03/09/2021 | 03/09/2021 |
| Initials | AZ | PM | TK |




EP00536 / E

| Material | | | |
|----------|-----------|---------|----------|
| No. | Plan | Article | Quantity |
| 24 | YP09023 | 2631754 | 1 |
| 25 | YP09024 | 2631854 | 1 |
| 26 | M6x35 | 1001496 | 4 |
| 27 | M6 DIN985 | 101680 | 4 |

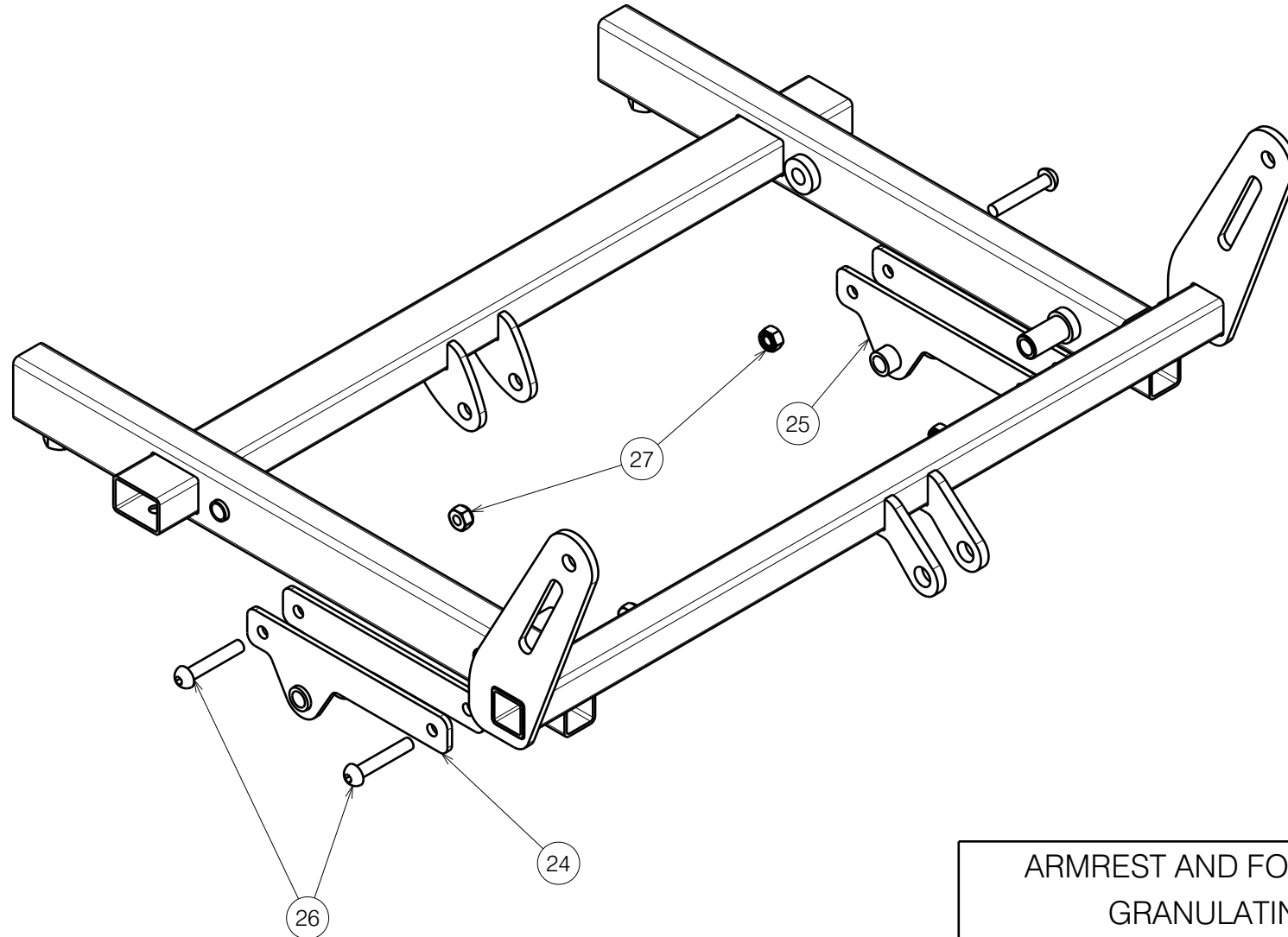

EP00536 / E

| | | | | |
|---|------------|------------|------------|---|
| ARMREST AND FOOTREST SUPPORTS / GRANULATING SYSTEM L70 Sheet 3/3 | | | | Plan No. |
| | | | | EP00536/ E |
| Scale: 1:9 | Drafted | Verified | Confirmed |  |
| Date | 03/08/2020 | 03/09/2021 | 03/09/2021 | |
| Initials | AZ | PM | TK | |