

Material			
No.	Plan	Article	Quantity
1	Soft	1907554	1,2m
2	Soft	1907554	0,25m
3	Velcro 50mm	1009792	2,2m
4	Velcro 100mm	2991789	C m
5	Velcro hook 100mm	2991788	C m
6	Foam 5mm	1990296	0,125m
7	Foam 20mm	2990519	0,125m



Size	35	40	45	50
A	360	410	460	510
B	270	320	370	420
C	220	270	320	370

\*on the second side


EP00495 / B

Section view A-A


BACKREST UPHOLSTERY L44 INOVYS				Plan No.
				EP00495/ B
Sheet 1/2				
Scale: 1:8	Drafted	Verified	Confirmed	
Date	6/03/2020	16/03/2020	16/03/2020	
Initials	PM	AS	TK	

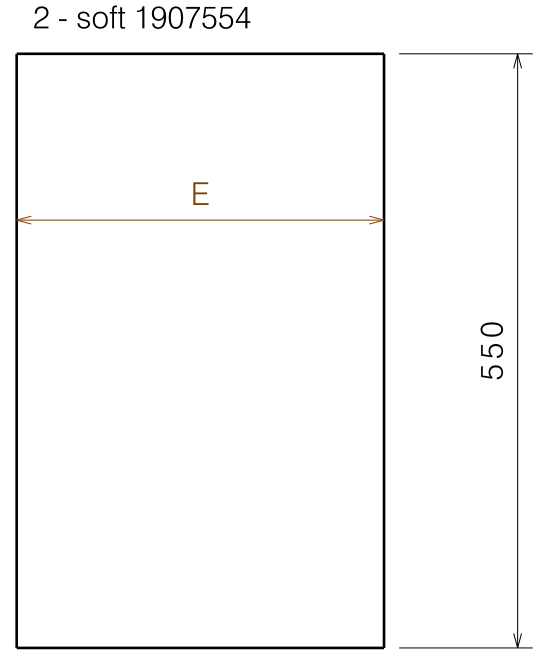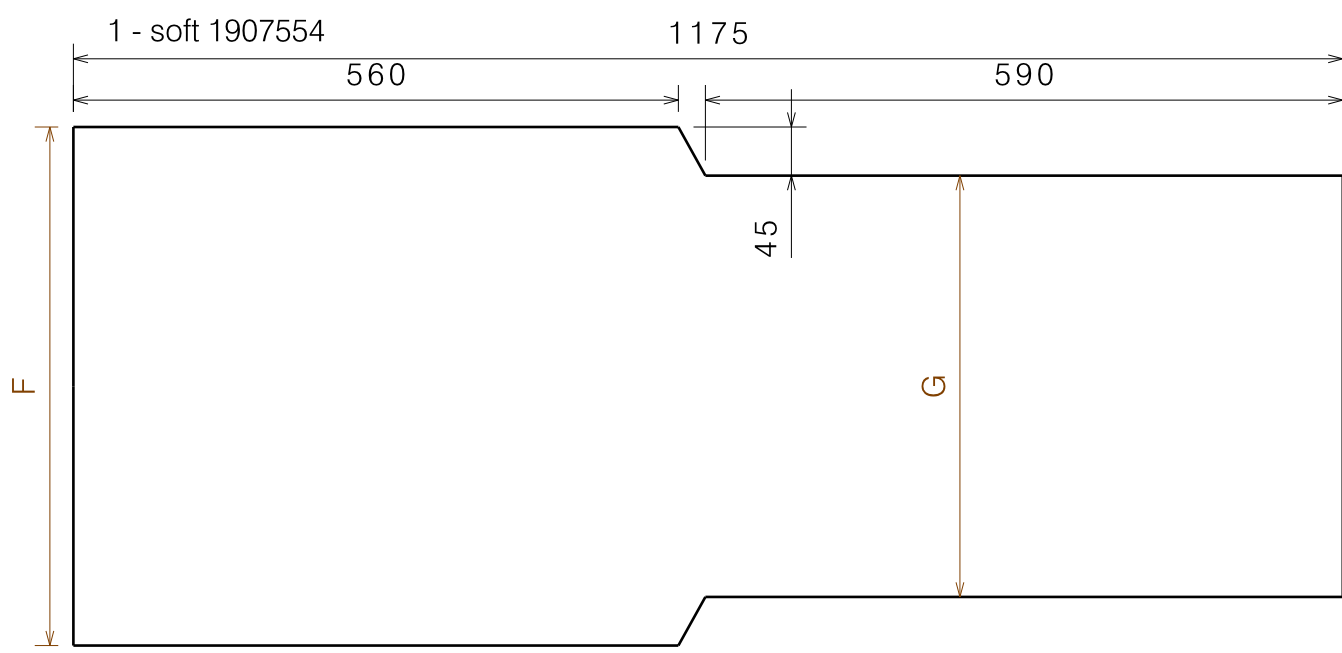


6 and 7 - foam 5+20mm - 1990296+2990519

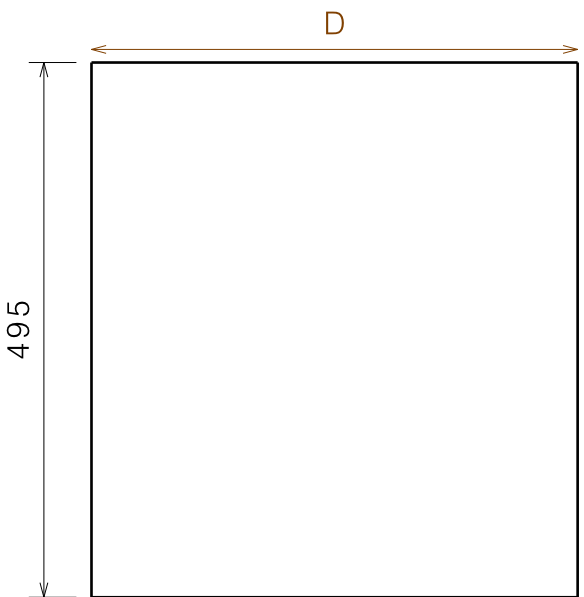


4- velcro 2991789

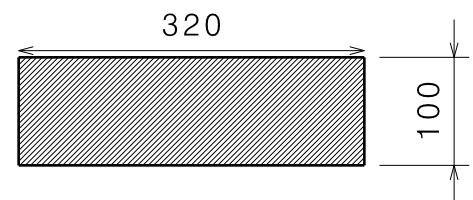

3- velcro 1009792

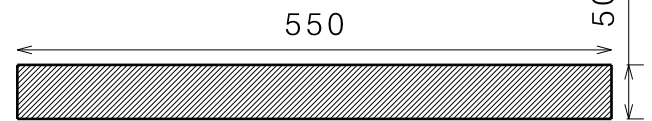

5- velcro hook 2991788

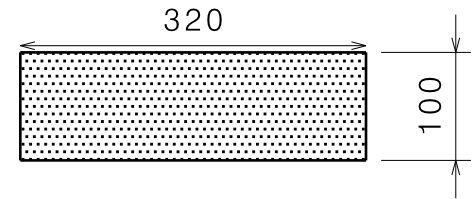

Size	35	40	45	50
D	350	400	450	500
E	240	290	340	390
F	380	430	480	530
G	290	340	390	440

EP00495 / B

BACKREST UPHOLSTERY L44 INOVYS				Plan No.
				EP00495/ B
Sheet 2/2				
Scale: 1:8	Drafted	Verified	Confirmed	
Date	6/03/2020	16/03/2020	16/03/2020	
Initials	PM	AS	TK	